

Disposable Germicidal Surface Wipes

McKesson disposable germicidal surface wipes are an ideal cleaning and disinfecting solution for hard, non-porous surfaces. They are made of non-woven cloth and contain a stable, low-pH-formulated disinfectant and deodorant. The wipes are pre-moistened and disposable. A self-threading dispenser is built into the lid to help ensure hassle-free dispensing. Active ingredients: N-alkyl (60% C14, 30% C16, 5% C12, 5% C18) dimethyl benzyl ammonium chloride and N-alkyl (68% C12, 32% C14) dimethyl ethyl benzyl ammonium chloride, and Isopropanol (21%).

Features and Benefits

- Durable, non-woven cloth material
- Nonabrasive; intended for use on nonporous surfaces
- Wipes are pre-moistened
- Designed for convenient dispensing and use
- Self-threading dispenser lid helps ensure economical single wipe dispensing



ORDERING INFORMATION

MFR #	DESCRIPTION	QTY
50-66160	Disposable Germicidal Surface Wipes, Large, 6" x 6.75"	160 wipes, 12/cs
50-66170	Disposable Germicidal Surface Wipes, X-Large, 9" x 12"	65 wipes, 12/cs
50-66000	Wall Bracket	12 ea/cs

Satisfaction Guaranteed

If you are not completely satisfied with any McKesson Brands product, you may return it for a full refund or credit.

Never Compromise on CareSM

With McKesson Brands, you'll never have to compromise. We deliver quality products at competitive prices, backed by our 100% satisfaction guarantee. We have decades of experience in the market, and we offer you support and work to help ensure you have the products you need, when you need them—making it even easier for you to provide the best care for your patients.

The product information contained in this document, including the product images and additional product materials, was collected from various supplier sources. All product claims and specifications are those of the product suppliers and have not been independently verified by McKesson Medical-Surgical or its affiliates ("McKesson"). McKesson is not responsible for errors or omissions in the product information.

The properties of a product may change or be inaccurate following the posting or printing of the product information in the document, either in the print or online version. Caution should be exercised when using or purchasing any products from McKesson's online or print documents by closely examining the product packaging and the labeling prior to use.

Due to product changes, information listed in this document is subject to change without notice. This information is placed solely for your convenience in ordering and McKesson disclaims all responsibility for its completeness and accuracy, whether or not the inaccuracy or incompleteness is due to fault or error by McKesson.

Wipes Hospital and General Disinfection or Bactericidal:

This product kills the following bacteria in two minutes on pre-cleaned hard, non-porous surfaces at room temperature when used as directed:

- *Acinetobacter baumannii* {ATCC 19606}
- *Escherichia coli* – with [(extended beta-lactamase resistance) [(ESBL)]] [(E. coli)] {ATCC BAA-196}
- *Enterobacter aerogenes* {ATCC 13048}
- Carbapenem-resistant *Klebsiella pneumoniae* [(CRKP)] {ATCC BAA-1705}
- *Listeria monocytogenes* {ATCC 19117}
- Methicillin-Resistant *Staphylococcus aureus* [(MRSA)] {ATCC 33591}
- *Mycobacterium bovis*, BCG [(tuberculosis)][(TB)] {Organon Teknika Corp}
- *Pseudomonas aeruginosa* {ATCC 15442}
- *Salmonella enterica* {ATCC 10708}
- *Serratia marcescens* {ATCC 14756}
- *Staphylococcus aureus* {ATCC 6538}
- Vancomycin – Resistant *Enterococcus faecium* [(VRE)]{ATCC 51559}

Wipes Virucidal*

This product kills the following viruses* in two minutes on pre-cleaned hard, non-porous surfaces at room temperature when used as directed:

- *Avian Influenza A Reassortant Virus* [(H3N2)] {ATCC VR-2072}
- *Rotavirus* [Strain WA]
- *Hepatitis B Virus** [(HBV)] [Strain LeGarth]
- *Hepatitis C Virus** [(HCV)]
- *Herpes Simplex II Virus* [(HSV 2)] [G Strain]
- *Human Immunodeficiency Virus** [(HIV-1)] [(associated with AIDS)]
- *Swine Influenza A Virus* [(H1N1)] [Strain A/Swine/Iowa/15/30] [(formerly called “swine flu”)] {ATCC VR-333}
- *Influenza A Virus* [Strain A2/Hong Kong] {ATCC VR-544}
- *Rhinovirus* [Type 14] [RV-14] {ATCC VR-284}

Wipes Fungicidal

This product kills the following fungi in two minutes on pre-cleaned hard, non-porous surfaces at room temperature when used as directed:

- *Trichophyton interdigitale* [(formerly known as *Trichophyton mentagrophytes*)] [(ringworm)] [(Athletes Foot)] {ATCC 9533}

McKesson Medical-Surgical
9954 Mayland Drive
Richmond, VA 23233
mms.mckesson.com/mckesson-brands

Post-Acute Care and Home Health
800.347.2456

Home Medical Equipment
888.822.8111

Primary Care
866.625.2679

McKesson Brands Support
800.777.4908