

KaiBiLi™


Saliva Collection Kit




Saliva Collection Kit (Dry)

Saliva Collection Kit (NTM*)

*NTM: Nucleic acid Transport Medium

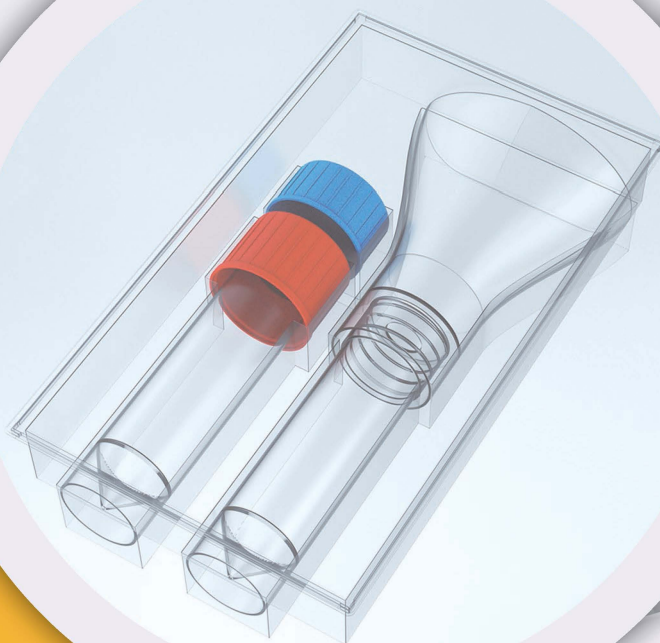

Dimensions

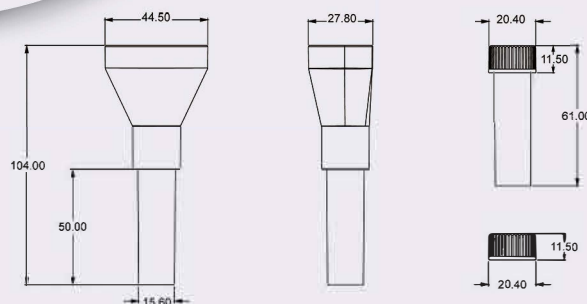

DISTRIBUTED BY CLIWAIVED, INC.

ADD : 2721 Loker Avenue W Carlsbad, CA 92010

TEL : 1-888-882-7739

FAX : 801-720-7568

Web : <https://cliawaived.com/>

EC REP

Lotus NL B.V.

Koningin Julianaplein 10, le Verd, 2595AA, The Hague, Netherlands.

KaiBiLi

Saliva Collection Kit (Dry)

- Saliva collection tube(Preservation tube) & Funnel & Screw cap
- Funnel designed to fit mouth shape for easy saliva collection.
- The collection process is easy, fast and painless. No injury or discomfort to the human body.
- Tightly closed lid with specimen bag can limit the possibility of specimen leakage.
- Tube designed with unique barcode and user labeling area.
- No DNase/RNase
- EO sterilization



Saliva Collection Kit (NTM)


- Saliva collection tube(Preservation tube) & Funnel & Screw cap & NTM solution
- The 5 mL tube is filled with 2 mL NTM solution.
- The non-invasive method to capture saliva samples for DNA/RNA extraction.
- The Saliva Collection Kit ensures sample stability during storage and transport at ambient temperatures.
- This preservation solution has good antibacterial performance and high storage efficiency and effectively inactivates pathogens (e.g. virus) in collected samples.
- Samples in buffer can be frozen (-20/-80°C) for prolonged periods.



Order information

Cat. No.	Product	Pkg
A211112	Saliva Collection Kit(Dry); Tube size: 60x15mm	1
A211113	Saliva Collection Kit(Dry); Tube size: 102x15mm	1
M221131	Saliva Collection Kit(NTM); Tube size: 60x15mm, NTM: 2mL	1
M221132	Saliva Collection Kit(NTM); Tube size: 102x15mm, NTM: 2mL	1

Saliva Collection Kit (NTM) Operation Instructions

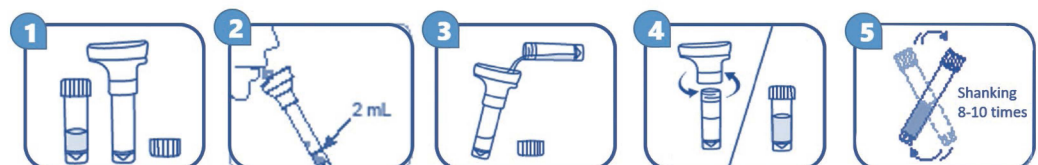

1. Remove the kit from packaging.
2. Deep cough and spit into the saliva collector, up to 2ml marker.
3. Add the preservation solution pre-filled in tube.
4. Remove the saliva collector and screw the cap.
5. Invert tube to mix, shaking 8-10 times.