

# HIV 1/2 STAT-PAK<sup>®</sup> Assay



## Performance Matters

An FDA approved, rapid point-of-care assay for the detection of HIV-1 and HIV-2 antibodies in fingerstick or venous whole blood, serum or plasma.

### Superior Performance

- The only HIV test that exceeds WHO performance thresholds<sup>1</sup>
- Sensitivity: **99.7%**; Specificity: **99.9%**<sup>2</sup>

### Innovative Design

- Small sample volume: **5 µl**
- Built-in procedural control

### Easy to Use

- Test in 3 easy steps: **Prepare, Run, Read**
- Requires minimal training


<sup>1</sup> Kosack CS et al. Journal of the International AIDS Society 2017, 20:21345  
<sup>2</sup> Refer to product insert for details

# Test for HIV in 3 Easy Steps with STAT-PAK®

Please refer to the complete product insert provided in the test kit

## 1 Prepare

Transfer sample to Sample Well (S)

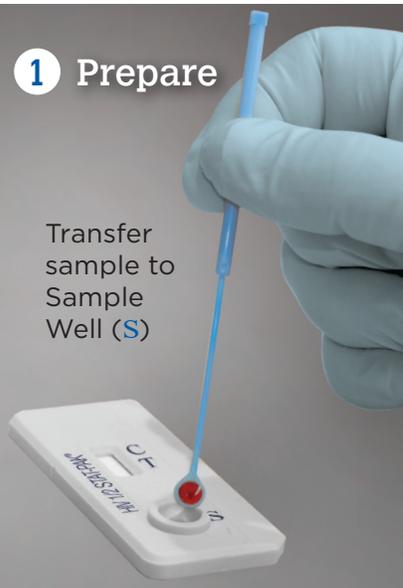


## 2 Run

Add 3 drops of running buffer to Sample Well (S)



WAIT  
**15**  
MINUTES

## 3 Read

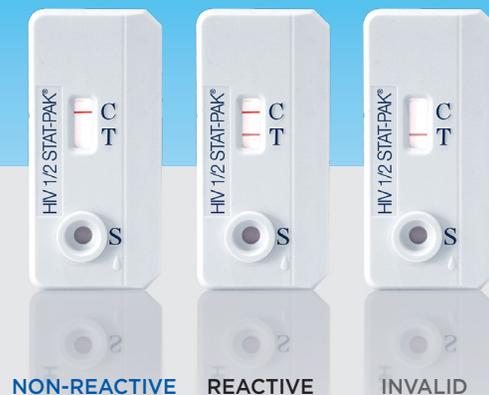

## Product Information

Information Type	Product Detail
Sample Volume	5 µl
Method	Lateral Flow
Test and External Controls Shelf Life*	24 months
Storage Conditions	8 - 30°C (46-86°F)
Time to Results	15 minutes

## Ordering Information

Product	Catalog Number
HIV 1/2 STAT-PAK® Assay FDA Approved, CLIA Waived	60-9505-1
HIV Rapid Test Control Pack	60-9549-0

\* From date of manufacture

Reimbursement Codes: 86703 or 86703QW  
G0435 (for Medicare)

## For More Information

Visit: [www.chembio.com](http://www.chembio.com)  
Call: 1.844.CHEMBIO  
Email: [info@chembio.com](mailto:info@chembio.com)