



AlcoScreen

Alco-Screen is a rapid, highly sensitive method to detect the presence of alcohol in saliva and provide an approximation of relative blood alcohol concentration. Alco-Screen is a simple, one-step, 2-min. test requiring no instrumentation, or training

CLIAwaived.comTM

For more information please contact CLIAwaivedTM Inc. at:

TEL: (858) 481-5031 TOLL FREE: (888) 882-7739

E-MAIL: info@cliawaived.com WEB: www.cliawaived.com

Distributed by: CLIAwaivedTM Inc., San Diego, CA

Alco-ScreenTM Instructions



1. Insert test pad into mouth and saturate with saliva. Remove test pad and wait two minutes.



2. At two minutes, compare test pad with the color chart printed on the back panel of foil package. Within 15 seconds choose the closest color match to estimate the alcohol concentration.