

# SWAB

# STAT<sup>+</sup>SWAB



## STATSWAB™

The STATSWAB™ is a NEW and INNOVATIVE, fully integrated oral fluid drug screen device. The STATSWAB™ is the simplest and fastest oral fluid drug screen on the market. Simply swab the mouth and screw the cap on. It's that easy! See back for details.

*For Forensic Use Only.*

## CALL FOR A FREE SAMPLE!

### TEST FEATURES

- Fully Integrated
- Simple Mouth Swab
- No Steps to Activate
- Observable
- No Restroom Needed
- Test Up to 11 Drugs
- Fast
- Accurate
- Affordable
- Transport Sample to Lab

### TEST AVAILABLE

#### Drugs of Abuse:

Marijuana  
Cocaine  
Opiates  
Phencyclidine  
Methamphetamine  
Amphetamine  
Benzodiazepines  
Oxycodone  
Barbiturates  
Methadone  
Buprenorphine

# STAT<sup>+</sup> SWAB

## EASY-TO-USE

## FAST

## ACCURATE

CALL FOR A FREE SAMPLE!



### PROCEDURE

- Do not place anything in the mouth including food, drink, gum, or tobacco products for at least 10 minutes prior to collection of oral fluid specimen.


#### STEP 1

- Unscrew the Collector Cap to pull out the collection stick with Sponge from the Collection Chamber.



#### STEP 2

- Insert the sponge end of the collection stick into the mouth. Close mouth and actively move the sponge back and forth for saliva excretion. Gently pressing the sponge between the teeth and the tongue will assist in the sponge saturation process.

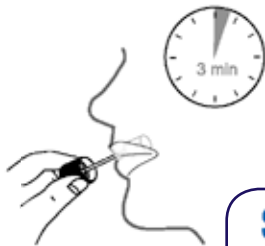

- Soak sponge into saliva in mouth and under the tongue for a total of 3 minutes until the sponge becomes completely saturated with saliva. No hard spots should be felt on the sponge when saturated.




#### STEP 3

- Remove the sponge from the mouth. Place the collection stick with saturated sponge into Collection Chamber.

### INTERPRETING RESULTS

- Peel off the label to read test results. Read results at 10 minutes. Do not read results after 1 hour.

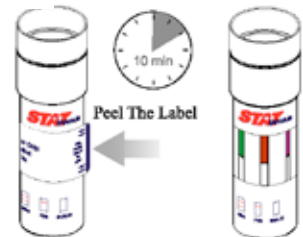

#### STEP 4

- Secure the cap and start the timer.



CLIAwaived.com

Distributed by: **CLIAwaived.com™**  
 San Diego, CA 92121  
 tel 858-481-5031  
 toll free 888-882-7739  
 www.cliawaived.com