


Status iFOBT

lifeSign

**Rapid Immunassay for the Qualitative Detection
of Immunochemical Fecal Occult Blood
(Human Hemoglobin in Human Fecal Specimens)**

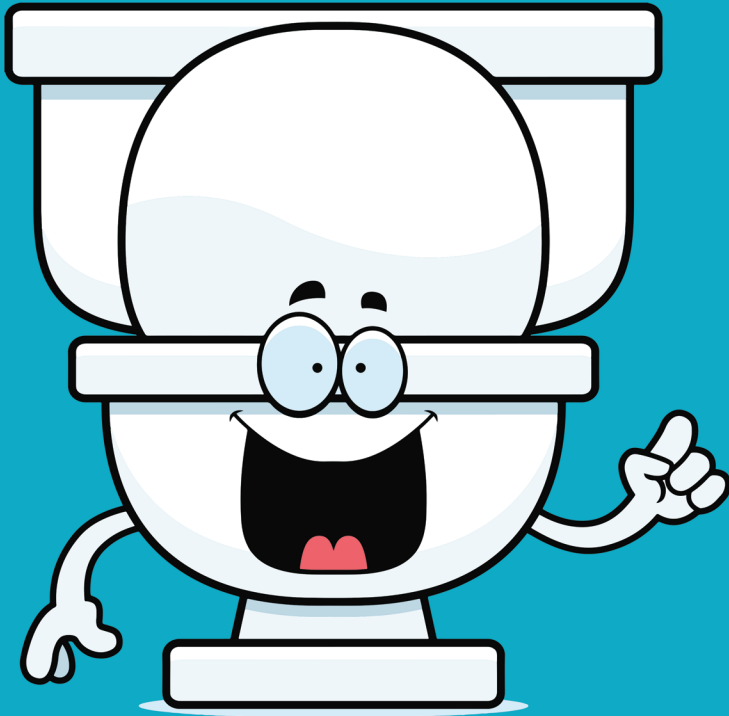

**Did you know colon cancer
is the second leading
cause of cancer deaths in
the United States?**

**Regular screening results
in early detection.**



- **CLIA Waived**
- Superior sensitivity and specificity
- Human hemoglobin specific reduces false positives
- No dietary or drug restrictions
- Easy one-step test procedure
- Specimen can be collected in office or at home
- Patient friendly and easy-to-follow instructions

Sample Collection

1. Deposit the fecal sample on collection paper or in a clean container.


NOTE: Do not allow collection paper or feces to come into contact with toilet water or urine. The sample should not be tested if it becomes wet or contaminated. If this happens, use a new collection paper and repeat the sample collection procedure.

2. Unscrew the sampler from the collection tube with the fecal sample still on collection paper, randomly insert the grooved end of the sampler into the fecal sample in at least 6 different sites. Ensure the grooved end of the sampler is completely covered with fecal sample.

3. Insert the sampler into the collection tube and firmly tighten it. Do not reopen.


4. Shake the tube vigorously to mix the sample and the collection tube.

NOTE: The extracted fecal sample may be stored at room temperature for up to 10 days.

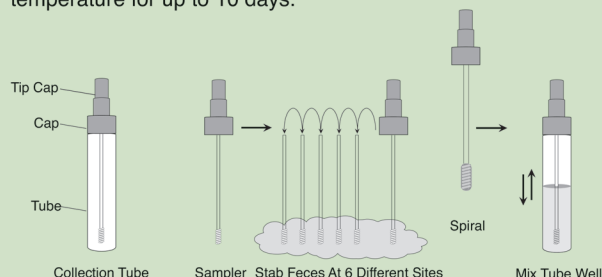


Sample Application

1. Shake the collection tube vigorously for 20-30 seconds to ensure the sample is well mixed with the buffer. Some small amount of sample may not dissolve; this is normal.
2. Remove test device from its foil pouch by tearing at the notch.
3. Tap the collection tube on a hard surface to dislodge any trapped air in top of cap.
4. Unscrew cap tip on the collection tube. Holding the tube vertically above the sample well of the **Status iFOBT** device, squeeze the tube gently to dispense three (3) drops into the sample well.

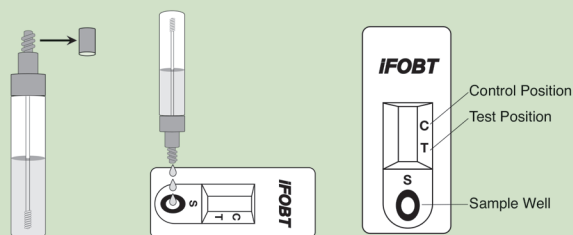

5. Read the result in 5-10 minutes. Important: DO NOT READ RESULTS AFTER 10 MINUTES.

Interpretation of Results

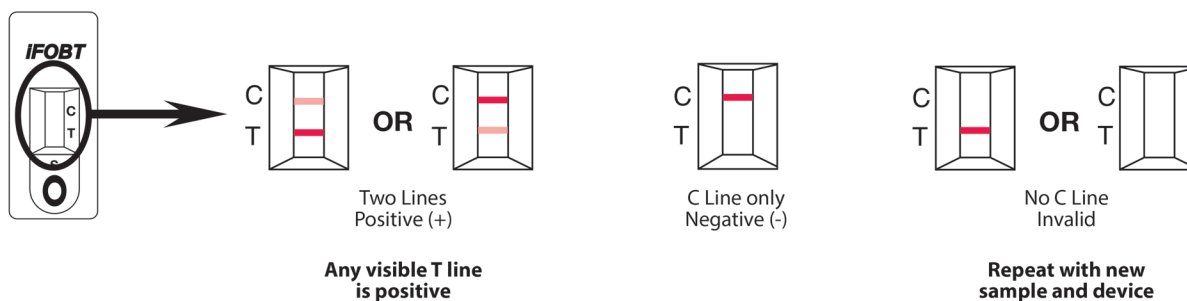

ORDERING INFORMATION

PRODUCT	# OF TESTS	ITEM #
Status iFOBT Complete Kit with Devices, Tubes, Patient Mailers	30	38030-DTM
Status iFOBT Cassettes and Tubes	50	38050-DT
Status iFOBT Sample Collection Tubes (refill)	50	38050-T
Status iFOBT Mailing Envelopes (refill)	50	38050-M
Status iFOBT Control Set - POS & NEG	2 x 0.5ml	200020



Proudly made in the USA

For more information or to place an order, contact:

lifeSign

A PBM Group Company

85 Orchard Road • Skillman • New Jersey • 08558
800.526.2125 • 732.246.3366 • Fax: 732.246.0570 • www.lifesignmed.com

CPT CODE

82274QW

G0328QW

(Medicare Annual Screen)