

CLIAwaived,™ Inc. RAPID TSH CASSETTE



Developed by doctors, this test determines elevated thyroid stimulating hormone (TSH) in whole blood, the first clinical indicator of hypothyroidism. This rapid immunoassay is ideal for physicians' office labs, pharmacy programs and health promotions. Determining hypothyroid patients in your office allows you to make timely decisions, saving money for both you and your patients. The test requires a drop of blood from a finger stick or venous draw, with results in minutes. It is easy-to-use, cost-effective and accurate.

- Test patient on first visit
- Results in 10 minutes
- Room temperature storage

THYROCHEK-120-A-KIT (20 cassettes)

CPT CODE: 84443QW


National limit Amount: \$23.47




RAPID TSH CASSETTE

The **CLIAwaived,™ Inc. Rapid TSH Cassette** is an immunoassay for the qualitative determination of human thyroid stimulating hormone (TSH) in whole blood samples. This test is intended to detect TSH at concentrations > 5mIU/L. It is intended for use by medical professionals to screen an ambulatory adult population for primary hypothyroidism.


- 1**




Remove test cassette and pipette from foil pouch by tearing at notch at corner of pouch. Place the cassette on a hard flat surface with the windows facing up.
- 2**



Add one drop of whole blood directly into circular specimen well "S" located in the lower portion of cassette with pipette provided in the pouch. Discard pipette into waste container after use. Set timer and wait 90 seconds before proceeding.
Add 4 full drops of buffer into the circular specimen well "S" located at the bottom of the cassette.
- 3**

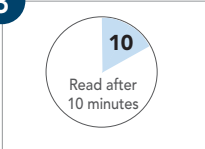


Set timer for 10 minutes. Do not move the cassette during this time.
- 4**



At the end of 10 minutes, read the line(s) in the rectangular results window of the cassette. Do not move the cassette until you have checked the lines. Do not read results after 15 minutes.

 Please refer to package inserts for detailed instructions and technical information.

DISCLAIMER – The CPT information contained in this document is provided as representative examples of reimbursement in this category. It is intended to assist providers in accurately obtaining reimbursement for health care services. It is not intended to increase or maximize reimbursement by any payor. Providers should consult their payor organizations with regard to local reimbursement policies. The information provided in this document is for information purposes only and represents no statement, promise or guarantee by CLIAwaived™ Inc. All CPT codes are supplied for information purposes only and represent no statement, promise or guarantee by CLIAwaived™ Inc. that these codes will be appropriate or that reimbursement will be made. CPT codes and descriptions are copyrights of the American Medical Associations. CPT does not include fee schedules, relative values or related ratings. The source for this information is the Center for Medicare and Medicaid Services. The content provided by the Center for Medicare and Medicaid Services is updated frequently. It is the responsibility of the service provider to confirm the appropriate coding required by their local Medicare carriers, fiscal intermediaries and commercials. All product information and prices are subject to change without notice at any time. Please call our office before ordering to verify current prices.

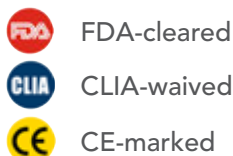

CLIAwaived™ Inc.

For more info, please call: **888.882.7739**

Visit us online at: **www.cliawaived.com**