

FENTANYL RAPID TEST

Fentanyl Urine Test Cassette is competitive binding, lateral flow immunochromatographic assay for qualitative detection of Fentanyl in human urine at the cutoff concentrations of 1 ng/mL.

The test provides only preliminary test results. A more specific alternative chemical method must be used in order to obtain a confirmed analytical result. GC/MS or LC/MS is the preferred confirmatory method.

For in vitro diagnostic use only.

- High Accuracy
- Easy Specimen Collection
- Fast Results in 5 Minutes
- Convenient Procedure
- Easy Visual Interpretation
- Lower Cut-Off (1ng/ml)

Fentanyl Rapid Test is a rapid urine screening test that can be performed without the use of an instrument. Fentanyl Rapid Test yields a positive result when Fentanyl in urine exceeds 1 ng/mL.


Product Description	Type	FDA	Packaging	Catalog #
Fentanyl Rapid Test (Urine)	Cassette	■	25 tests	CLIA-RDTD-FEN-1

CLIA Classified as Waived



FENTANYL RAPID TEST


Fentanyl Urine Test Cassette is competitive binding, lateral flow immunochromatographic assay for qualitative detection of Fentanyl in human urine at the cutoff concentrations of 1 ng/mL.

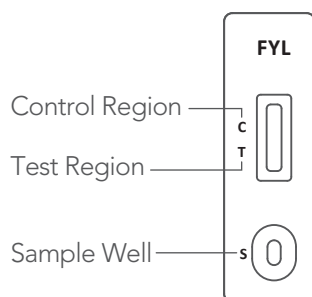

Allow the test to reach room temperature before testing.
Wash your hands thoroughly.


1. Collect urine in a clean and dry container.
2. Open the foil pouch, remove the test cassette and dropper, discard the desiccant. Place the test cassette on a clean and level surface. Hold the dropper vertically and transfer 3 full drops of urine to the specimen well next to the letter "S" on the test cassette. Do not spill urine into the test window. Start a timer.
3. Read the test results at 5 minutes, do not read results after 10 minutes.

• Collect urine.

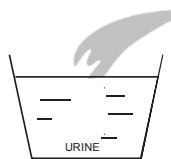


1

• Add 3 drops of urine.

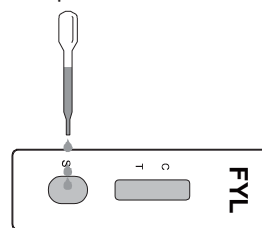

2

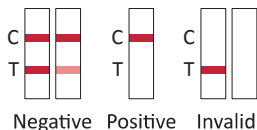

3

FDA FDA-cleared

CLIAwaivedTM Inc.

For more info, please call: **888.882.7739**

Visit us online at: **www.cliawaived.com**

Rev.1.11.23