




Rapid Test Cup "RTC" + Fentanyl Procedure Card

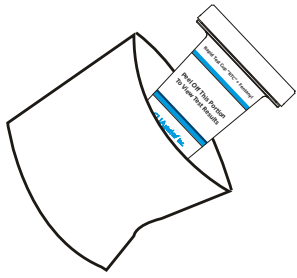


1. Remove the cup from the foil pouch.

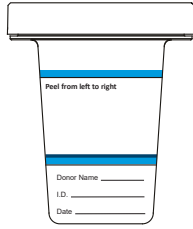

2. Write donor ID and date on the label.

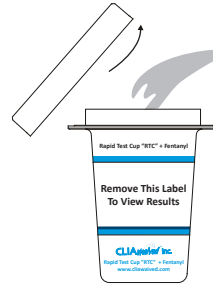

3. Collect specimen in the cup.

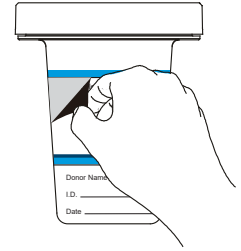

4. Peel off the label and read results. The adulteration strip(s), if applicable, should be read between 3-5 minutes. Read the drug test result at 5 minutes. Do not read after 10 minutes.

INTERPRETATION OF RESULT

Each strip contains one drug test. C region shows validity of a test result. T region shows result for this test.

For C region:

The appearance of a line indicates a valid result.


No line means an Invalid result.

If a test strip does not have a line in the C region, test results are Invalid for this strip.

For T region:


The appearance of a line indicates a Negative result.

Note: Any test line, even a very faint test line, is considered a negative result.

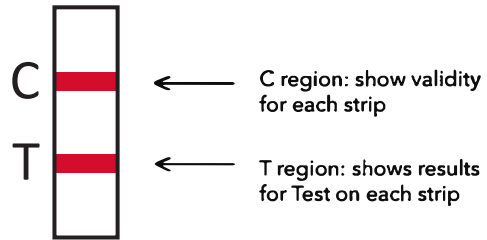

No line indicates a Preliminary Positive result.

Note: Any urine with preliminary positive results should be sent to a laboratory for confirmation.

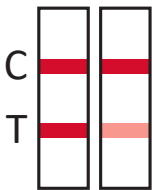


(Negative Result)

A red or pink line next to the "T"(drug test line) under the drug name indicates a negative result for that drug. Certain lines may appear lighter or thinner than other lines.



(Preliminary Positive Result)

If NO red or pink line appears next to the "T" under the drug name, the sample may contain that drug.

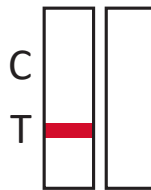

(Invalid Result)

A colored line should always appear next to the letter "C" on every test strip. If no control line appears on any of test strips, the result is invalid.

	No adulteration	Adulteration
AD1 (OX)		
AD2 (S.G.)		
AD3 (pH)		
AD4 (NIT)		
AD5 (GLU)		
AD6 (CRE)		

Read adulteration test results at 3 - 5 min. 143026902

Semi Quantitative results are obtained by visually comparing the reacted color blocks on the strip to the printed color blocks on the color chart.

CLIAwaived,™ Inc. Rapid Test Cup "RTC" + Fentanyl



DETECTS UP TO 14 DRUGS

Amphetamine (AMP)
Buprenorphine (BUP)
Secobarbital (BAR)
Oxazepam (BZO)
Cocaine (COC)
Methamphetamine (MET)
Methylenedioxymethamphetamine (MDMA)
Morphine (OPI 300)
Methadone (MTD)
Oxycodone (OXY)
Phencyclidine (PCP)
Nortriptyline (TCA)
Marijuana (THC)
Fentanyl (FYL)

+ adulteration
(CRE/OX/pH)

- Fast, dark test lines
- FDA-cleared and CLIA-waived
- Non-negative results in five minutes
- Donor-friendly cup opening
- Results can be photocopied
- Optional tests for adulterants
- Remarkable pricing and quality

Call your sales representative now at 1-888- 882-7739