

RAPID DIAGNOSTICS

osom[®]

iFOB Test

FOR THE DETECTION OF FECAL OCCULT BLOOD/HUMAN HEMOGLOBIN



www.osomtests.com



FEATURES:

- ✓ **ACCURATE;** Specific to human hemoglobin
- ✓ **FAST;** Results in 5-10 minutes
- ✓ **SAMPLE;** Easy three step procedure with <1 minute hands on time
- ✓ **IMPROVE PATIENT COMPLIANCE;** No dietary or drug restrictions
- ✓ **CUSTOM CARE;** Technical support staffed with certified medical technologists


RAPID DIAGNOSTICS

www.osomtests.com


OSOM[®]

iFOB Test


TEST PROCEDURE / RESULTS

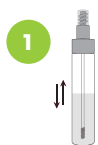

Shake tube vigorously for 20-30 seconds to ensure sample is well mixed.

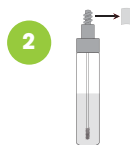

Unscrew tip cap on the stool collection tube.

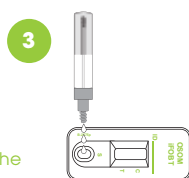

Squeeze tube gently to dispense three (3) drops into the sample well.




Read the result in 5-10 MINUTES



(examples of positive results)
Positive



(example of negative result)
Negative

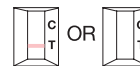

(examples of invalid results)
invalid

DESCRIPTION	CONFIGURATION	CATALOG #	STORAGE/CPT	CPT CODE
OSOM IFOB TEST	25 TESTS	1002	ROOM TEMP	DIAGNOSTIC CODE 82274QW SCREENING CODE G0328QW
OSOM IFOB POSITIVE & NEGATIVE CONTROLS	2 VIALS (1 POS; 1NEG)	1000	ROOM TEMP	
OSOM IFOB PATIENT COLLECTION MAILERS	25 MAILERS	1003	ROOM TEMP	
OSOM IFOB STOOL COLLECTION TUBES REFILL	25 TUBES	1001	ROOM TEMP	

OSOM[®] is a registered U.S. trademark of Sekisui Diagnostics, LLC. © 2015 Sekisui Diagnostics, LLC. All rights reserved.