

The GM 550 has a large, easy-to-read LCD screen. The display is crystal clear and backlit so it can be read even in low light. The GM 550 also has a 500 test memory with 7, 14, 30 and 90 day averaging and it uses our patented, large-sized, rigid test strips, which add to its ease of use. With Bionime's noble gold nano-technology and unique auto-code feature, the GM 550 provides the most accurate results possible.



GM 550

- Proven accuracy
- Large, injection molded test strips
- Large, easy-to-read, backlit display
- 1.0 μL blood sample size required
- Auto coding
- Energy-saving auto-start & shut-off
- Ergonomic design
- 500 test memory with 7, 14, 30 and 90-day averaging
- Downloadable results



Insert strip in socket. The meter will calibrate automatically.



While blood drop symbol is flashing, apply a tiny blood sample.



Wait 5 seconds for test results to appear.



Take strip out of strip port without touching blood sample.



About Bionime

At Bionime, we're dedicated to manufacturing blood glucose monitoring devices with the highest possible levels of quality, accuracy and reliability. Bringing together top industry professionals in chemistry, software development, mechanical engineering and precision manufacturing, we've created an award-winning line of glucose meters that are affordable, easy to use, and proven accurate. It's time to take testing seriously.

BIONIME

Accurate intel for the war on diabetes.

Distributed by: **CLIAwaived.com™**
San Diego, CA 92121
tel 858-481-5031
toll free 888-882-7739
www.cliawaived.com